

# ***Baxter***




## How to get the most out of your Baxter Elastomeric Infusion System

PATIENT QUICK GUIDE



1

Ensure the **POSITION** of your Elastomeric Device is at the **SAME HEIGHT** as the connection:

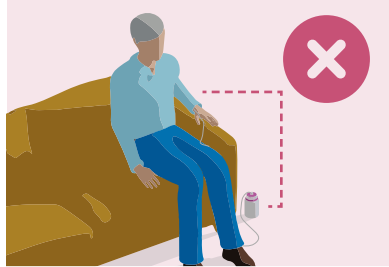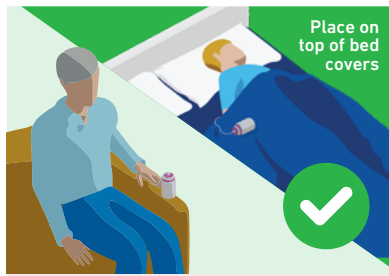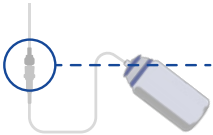

X



2

Check the line of your Elastomeric Device is **NOT OBSTRUCTED** or **KINKED**

✓



3

Ensure your  
Elastomeric  
Device is **NOT**  
**EXPOSED**  
to **HIGH**  
**TEMPERATURES**



Ideally placed in  
provided bag


Do NOT Place under pets



Do NOT place near heaters

4

Some moisture  
on tubing during  
shower OK.

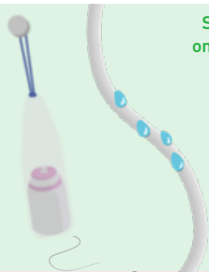

DO NOT  
Submerge  
Infusor



It is **POSSIBLE TO SHOWER** with your  
Elastomeric Device  
without compromising  
the line

# **Baxter**

**The devices pictured may not be an exact representation of your device. For more information about your Baxter Elastomeric Infusion System refer to your copy of *Patient Guide for Home Infusion Success*, or contact your healthcare provider with any additional product questions or concerns.**

## **Australia**

Baxter Healthcare Pty Ltd  
1 Baxter Drive  
Old Toongabbie NSW 2146  
1800 229 837  
baxterhealthcare.com.au

## **New Zealand**

Baxter Healthcare Ltd  
33 Vestey Drive  
Mount Wellington Auckland 1060  
0800 229 837  
baxter.co.nz

**DOWNLOAD THE  
PATIENT GUIDE HERE**



ANZ-MD78-230002

<https://athome.baxter.com/infusors>